



VINTEKS SAN. VE TIC. LTD. ŞTİ.  
BAĞLAR MAH. OSMANPAŞA CAD. NO:56 34212  
GÜNESLİ İSTANBUL/TURKEY

## MATTRESS CLEANING INSTRUCTIONS

### General Instructions:

Clean by neutral water (with soap).  
Use a damp sponge with solution to clean the surface of the product. When cleaned use a dry clean cloth to wipe over the surface.

If necessary, the listed disinfectants could be used as long as the corresponding concentration below is applied and the product is rinsed after cleaning.  
If there is a new disinfectant, it needs to be tested by Vinteks.

### List of **APPROVED** disinfectants and cleaning agents suitable for PU Mattress Cover

PRODUCTS	CONCENTRATION	RESULT
ACTICHLOR PLUS	1.000 ppm	Ok
ANIOS DDHS	pure	Ok
DURR DENTAL	pure	Ok
CHLOR-CLEAN TABLETS	1.000 ppm	Ok
DETTOL	5%	Ok
DISMOZON PUR	4%	Ok
ETHANOL	%70	Ok
FORMALDEHYDE	%37	Ok
GERMATOL	5%	Ok
HANSAMED (spray)	pure	Ok
HAZ-TABS	1.000 ppm	Ok
HIBICET	pure	Ok
INCIDIN ACTIVE	2%	Ok
INCIDIN EXTRA N	2%	Ok
INCIDIN FOAM	pure	Ok
INCIDIN PLUS	2%	Ok
KODAN TINKTUR FORTE	pure	Ok
MEDICARINE	1.000 ppm	Ok
MIKROBAC FORTE	4%	Ok
MINUDES	pure	Ok
NATRIUMHYPOCHLORITE	0,1%	Ok
OCTENISEPT (spray)	pure	Ok

SAGROTAN (spray)	pure	Ok
SAGROTAN MED (spray)	pure	Ok
SANIT P20	pure	Ok
SOFTASEPT N (spray)	pure	Ok
SPRING	pure	Ok
SPRINT 200	2%	Ok
STERILLIUM	pure	Ok
SUMA BAC D10	1%	Ok
SURFA'SAFE	pure	Ok
ULTRASAN	pure	Ok
VIRKON	10 g/l	Ok
VOLVONE	40%	Ok
HYDROGEN PEROXIDE	1.000 ppm	Ok

List of **NON-APPROVED** disinfectants and cleaning agents suitable for PU Mattress Cover

PRODUCTS	RESULT
CLEAR SOL	Not Ok
HYCOLIN	Not Ok
LYORTHOL	Not Ok
SEKURAL SOFT	Not Ok
TURBO BREAK	Not Ok